



Accessory- Spoon

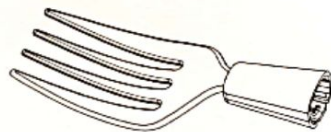

Accessory-Fork* (Optional)

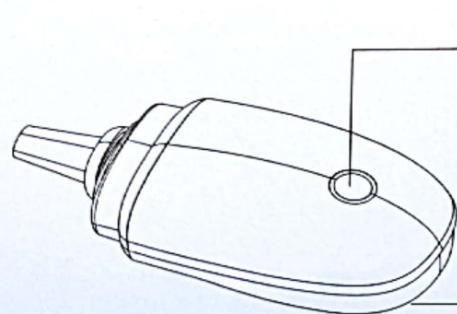


The Handle of GYENNO Spoon Lite


The Main Operation Indicator



Accessory-Smart Battery

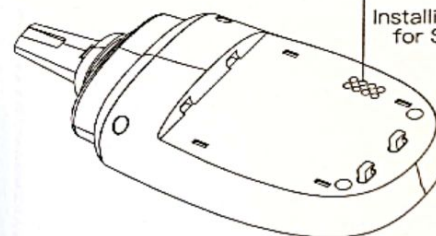

Installing Touch Pin for Smart Battery

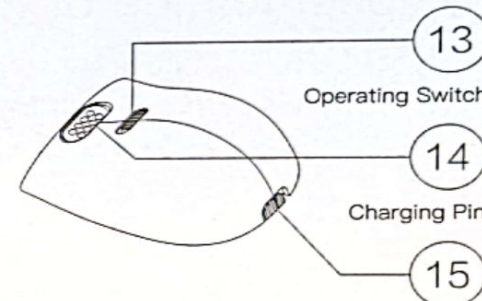

Operating Switch

Charging Pin

The Assembling and De-Assembling Switch of Smart Battery



USB Charge Cable

* For accessories (including a spoon, a fork, and a smart battery), you can purchase them additionally if needed.

I The specification of indicator

The main operation indicator of GYENNO Spoon Lite

Indicator color	Status
Red	Low battery
Green	Stand-by
Blue	In anti-shake

Indicator on smart charging storage box

	Color	Status
Charging Indicator ⑪	Red	Charging
	Blue	Fully charged
Power/Signal Indicator ⑫	Yellow	Power on
	Green	N/A

I How to use

1. Take out GYENNO Spoon Lite and turn on ⑩ operating switch. Hold the GYENNO Spoon Lite with hand while eating in the way you feel comfortable.
2. If ⑪ the main operation indicator is in green, it is standing by.
If the indicator is in blue, it is in the process of anti-shaking.
If the indicator is in red, its battery is too low, please recharge.
3. Please detach the accessory (spoon/fork) from handle to clean:
 - a) Please turn off the handle before you detach accessories. Never submerge the handle or hold it under running water.
 - b) You can clean the handle by antibacterial wipes or damp sponge with mild detergent.

I Charging

Charging will take your almost 4.5 hours

Please charge prior to use the product for the first time or the battery is going to run out. Charge the smart battery and it needs about 4.5 hours to be fully charged.

1. Put ⑩ the handle of GYENNO Spoon Lite into ① the main storage space, making sure the contact is good between the ⑭ charging pin of the handle and of the box.
2. Take out ⑫ USB charge cable and insert one end into the ② charging cable connection port.
3. The other end of ⑫ USB charge cable is inserted into the USB port. (5.0V---2.0A).

*If the ⑬ charging indicator is not on, it means the contact is not good. Please take out ⑩ the handle and put it in again. If the charging indicator is in red, it means it is charging now. And if the charging indicator is in blue, it means it is fully charged.

*No matter the handle of GYENNO Spoon Lite is on or off, it can be always put into the smart charging storage box. But we suggest to turn it off before charging.

I Other tips for using GYENNO Spoon Lite

1. Please relax while using GYENNO Spoon Lite, since anxiety will increase tremor. You need to adapt this new eating habit with GYENNO Spoon by a few days. For the first time, the user would be better to eat by yourself without others watching.
2. At the beginning, you may change various positions of holding GYENNO spoon Lite and adjust the angle and orientation of eating proactively to find the best way of your own usage.
3. Since you has to relearn how to eat, we suggest that it is no less than one week for you to test the effects and adapt to the new eating habit.
4. Please use a big and shallow bowl, so that there are enough space and angles for the use of GYENNO Spoon Lite.
5. Don't push your arm to hold the product. Try to relax, and let the arm shake freely, then GYENNO Spoon will finish its work.

I Precautions

1. The accessories (spoon/fork) of GYENNO Spoon Lite can be washed separately only after being detached from the handle. In addition, any other parts of GYENNO Spoon Lite cannot be completely sealed, so please pay attention to water and oil proofing.
2. GYENNO Spoon Lite can be used only for auxiliary feeding, not for other purposes, especially not for carrying too much weight or strength, which will reduce the lifecycle of the device and even lead to its damage.
3. As GYENNO Spoon Lite consists of motors and high precision components, please keep it away from the magnetic field environment. Long-time magnetic interference will affect its performance and lifecycle.
4. Please turn off ⑩ the handle after meal and remove the battery when you don't use it for a long time.
5. Whether GYENNO Spoon Lite is power on or off, please do not use external force to operate ① the spoon/ ② fork, which will be likely to reduce the lifecycle of the device and even lead to its damage.
6. Please prevent ① the charging box from water or other liquid.
7. Please don't hold ⑩ the handle under running water.
8. If ① the charging box is dropped into water, please cut off the power immediately.
9. Please prevent the device from fire or hot source, and from exposing to extreme environment.
10. Any damage caused by neglect, misuse, accident or mistreatment is not covered by warranty.
11. Turn off the handle before detaching the accessory (spoon/fork).
12. When removing or replacing GYENNO Spoon tableware accessories, you should hold the white shell part of the accessory (spoon/fork) in a hand closely, and use the other hand to hold the removable front-end part of the handle of GYENNO Spoon. Pull both parts apart with two opposed forces to separate them easily.

Warranty

Warranty & Disclaimer

1. The warranty period of ❶ Smart Charging Storage Box and ❷ the Handle of GYENNO Spoon Life is 1 year.
2. ❸ Batteries are consumables, the warranty period is 6 months.

The following circumstance is not covered in warranty:

1. Damages caused by drop.
2. Immersed in the liquid or liquid permeating.
3. Exposed in abnormal or extreme environments.
4. Any disassemble not indicated in the user guide.
5. Any exterior damage.

Product certification

This device complies with part 15 of the FCC Rules.



Service

If you need any service or information, or have any question.

 Please visit official website
www.gyenno.com

 Customer Service E-mail
service@gyenno.com